

How to Sew a Bias Tape Surgical Face Mask with Flexible Nose

Supplies:

100% Cotton Fabric, Prewashed

Floral Wire 6# (optional)

Bias Tape

HOW TO SEW A FACEMASK



1

CUT FABRIC

Cut 2 pieces of Quilting Cotton 9" x 7".
Cut 2 pieces of Bias Tape 54"



2

PIN FABRIC

Place 2nd piece of fabric on top with right sides together. Pin along right and left side leaving top and bottom unsewn.



3

SEW FACE MASK

Sew right and left side of mask using a 3/8ths inch seam allowance. Turn right side out and press flat.



4

CREATE PIN TUCKS

Fold mask in thirds and press. Create 1/2" pintucks facing upwards using pressed markings as a guide.



5

ATTACH BIAS TAPE

Open bias tape & align raw edges of fabric together with bias tape along mask. Sew along crease. Insert wire along top. Close bias tape, pin upwards & sew shut.



6

TOPSTITCH

Sew along either end of the wire to hold in place. Sew along over the pintucks along the right and left side of mask using a 3/8ths inch seam allowance.

